



Form: A-823-03

Rev.: 1.0

Rev. Date: 03/26/24

Page: 1 of 1

Total Lift Bed™ 425-T/V5 Transport Checklist

Leaving patient room

- Lower bed on to wheels
- Unlock wheels on both sides
 - Place brake pedals in horizontal for “neutral” and lateral movements
- *Optional:* Max inflate the mattress, press alarm silence
- Unplug bed
- Raise side rails, secure power cords and hand pendant
- After exiting patient room/doorway with bed, turn Power Drive on
 - Place brake pedals in upward position for “steering control” with power drive
- Ease into throttle to go, release fully to slow/stop
- Plug bed in at destination (if able)
 - Leave bed on wheels if bed must remain unplugged

Upon return to patient room

- Turn off power drive
- Place brake pedal in neutral position to manually reposition bed in room
- Once in desired location engage manual brakes on wheels (both sides)
- Plug bed back in
- Ensure mattress turns back on and resumes previous settings
 - Verify patient weight and pressure setting
- Ensure scale box is powered on