

## VitalGo LAL Mattress Cleaning

### TLB LAL, Alternating Pressure (VG-TLB-AM4)

#### Air Mattress Control Unit

**WARNING:** BEFORE ATTEMPTING TO CLEAN THE CONTROL UNIT, TURN IT OFF AND DISCONNECT THE POWER CORD FROM THE POWER SOURCE.

**WARNING:** DO NOT HEAT, STEAM AUTOCLAVE, OR IMMERSE THE CONTROL UNIT IN LIQUIDS Control Unit:

1. Wear eye goggles and protective gloves before starting the cleaning procedure.
2. We recommend Using EPA approved Hospital Disinfectants.
  - a. *Note: A fresh spray bottle of fresh disinfectant / detergent solution should be prepared daily to clean the control unit.*
3. Prepare the required amount of solution by following the preparation instructions provided with the germicidal detergent /disinfectant solution.
4. Pour required amount into a spray bottle.
5. Use a brush or cloth to wipe off dust. If necessary, spray the exterior of the top and the bottom enclosures, power cord and the cord plug with the prepared disinfectant / detergent solution. Using a damp cloth, wipe down the sprayed surface cleanly.
  - a. *Note: Do not spray excess amount of solution on the control unit.*
6. Once the control unit is clean, wipe the unit, power cord, cord receptacle, and the cord plug with a clean dry cloth.

## VitalGo LAL Mattress Cleaning

7. After the cleaning operations are completed, remove and dispose the protective gloves appropriately. Wash your hands thoroughly with antibacterial soap.

### Air Mattress

1. Wear eye goggles and protective gloves before starting the cleaning procedure.
2. Follow steps 2 through 4 above to prepare disinfectant solution.
3. Using a damp cloth, wipe down the air cushions and the mattress base. Once the air cushions and the base are clean, wipe them down with a dry cloth.
4. Air cushions should be washed periodically; top sheet will require more frequent washing. Set wash cycle to "heavy load" with warm water. Once the water is full, add manufacturer- suggested quantity of laundry detergent and/ or standard hospital disinfectants. If the air cushions or the top sheet becomes soiled with human waste or blood, clean immediately by wiping down. Use hospital recommended laundry detergent and/ or disinfectant per THE cleaning agent's manufacturer's instructions. *Note: Use non-chlorine bleach detergent.*
5. Shake cushions gently to remove excess water from inside the air cushions. Dry the cushions/top sheet on the lowest settings on the dryer until completely dry.
6. Leave the mattress to dry in a cool, dry area for an hour before using or storing. If the mattress is not used immediately, roll the mattress and insert it into a plastic bag and store it in a storage area.
7. After the cleaning operations are completed, remove and dispose the protective gloves appropriately. Wash your hands thoroughly with antibacterial soap.

## VitalGo LAL Mattress Cleaning

### STORAGE

- Fold or roll the previously dried air mattress and place the mattress in a plastic bag. Cable tie the plastic bag to keep the mattress dust free. Cover and store the Foam Air mattress in a flat position.

### MATTRESS SPECS

Air cushions: 70 denier urethane-coated nylon, R.F. welded, liquid proof and washable. Cal.117 pass.

Base: 1000 denier cordura, liquid proof and washable. Cal.117 pass.

Top Sheet: 70 denier urethane-coated nylon, low friction, low shear force producing, breathable, liquid resistant and highly vapor permeable. Cal.117 pass.

Optional Low Air Loss Top Sheet has 3 layers; top layer is breathable nylon, middle layer is an air distribution spacer material, and the bottom layer is a water resistant nylon layer.

#### Description Inflated Dim. LxWxH / Weight

Overlay:	80"x36"x5" (203x89x13cm)	8 lbs. 3.6 Kg.
Mattress:	80"x36"x7" (203x89x17.78)	23 lb. 10.5 Kg.

For More Information please visit



A-750-10

Rev.: 1.0

Rev. Date: 11/16/17

Page: 4 of: 4

## VitalGo LAL Mattress Cleaning

[www.VitalGosys.com/DOWNLOAD](http://www.VitalGosys.com/DOWNLOAD)